




Seat (1)

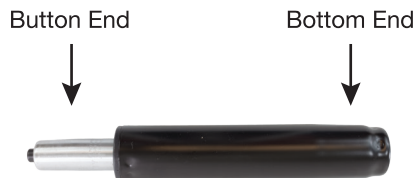

Pneumatic Cylinder (1)



5-Point Base (1)



Caster Wheels (5)



1. Remove protective cap from Pneumatic Cylinder. (Fig 1) Place the button end of the pneumatic cylinder into the opening in the metal plate on the bottom of the seat. (Fig. 2) Use minimal force to press the cylinder into the opening.
2. Turn the 5-point base upside down. Press one (1) caster into each of the ends of the 5-point base. (Fig. 3)
3. Turn the 5-point base over and place it on level ground so it is resting on all 5 casters. Place the bottom end of the pneumatic cylinder into the center opening in the 5-point base. (Fig. 4) Use minimal force to press the cylinder into the opening.
4. Carefully sit down on the seat\* until the cylinder locks into place.
5. Using the lever under the seat, you can adjust its height as needed.  
Height adjusts from 17.91" (45.5 cm) to 23" (58.5 cm). 5.25" (13.3 cm) total adjustability.

### ⚠ CAUTION:

- Maximum Weight Rating – 300 lb (136 kg)
- If components are damaged or missing, contact your dealer immediately.  
DO NOT use substitute parts.

\* You need to weigh at least 90 lb (41 kg) to adjust the height of the seat.

