



ASSEMBLY INSTRUCTIONS

STAIRCASE



ASSEMBLY INSTRUCTIONS/STAIRCASE

MAXIMUM WEIGHT CAPACITY 500 LBS

Thank you for counting on CanDo® to meet your physical therapy needs by purchasing this fine equipment. All of our equipment is designed and manufactured to provide years of dependable service because we want you as a customer now and in the future. Below are easy to follow instructions on how to properly assemble your equipment. Please contact us, if you experience any difficulty with the assembly or have questions.

INTENDED USE:

CanDo® Staircase is specifically designed and engineered for physical therapy use only. Persons using the staircase should be observed by trained medical/physical therapy personnel. Never leave anyone on the staircase unattended. Combination of weight and force exerted on the staircase during therapy is not to exceed a total of 500 lbs. Using this equipment for unintended purposes could result in damage to the equipment and serious injury.



Contents

- A. "A" Upright Post
- B. "B" Upright Post
- C. "C" Upright Post
- D. "D" Upright Post
- D. "E" Upright Post
- D. "F" Upright Post
- G. Platform
- H. Left Handrail
- I. Right Handrail

Hardware

- (12) 2¹/₄" Hex Bolts
- (24) 1" Washers
- (12) ⁹/₁₆" Locknuts

Tools Required


- Crescent or Adjustable Wrench
- Ratchet
- Phillips Screwdriver

Time Required

- 20 – 30 Minutes

1

Place platform(s) on side. Using two persons, position left side of staircase (pre-attached handrails and uprights) on left side of staircase and line up holes.

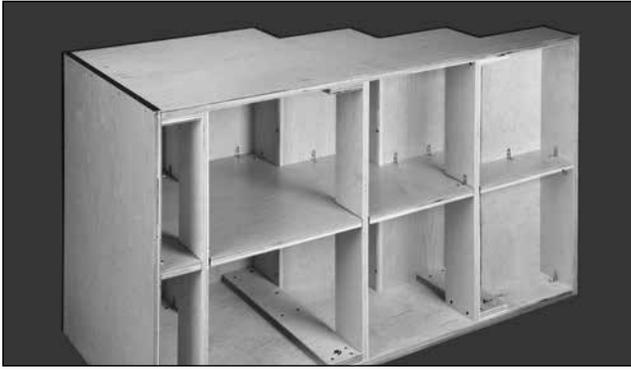

2

Attach all uprights with 2-1/4" hex bolts using a washer under the bolt head and a washer against the locknut. Tighten securely with 9/16" socket wrench and 9/16" crescent wrench.



3

Repeat steps 1 & 2 for the right side of the staircase.

4

Clean with disinfectant according to Cleaning Guidelines prior to use.

FOR SAFETY PURPOSES, TAKE TIME TO CHECK ALL BOLTS AND NUTS TO INSURE THAT THEY ARE SECURELY TIGHTENED AND THAT THE ENTIRE STAIRCASE HAS BEEN PROPERLY ASSEMBLED ACCORDING TO THESE INSTRUCTIONS. INSPECT FASTENERS PERIODICALLY.

CARE & CLEANING

CanDo® products are built to be durable. With proper care, your equipment will provide years of superior looks and performance. Clean staircase with a soft cloth and any mild cleaning agent that may be required for sanitation. NEVER use abrasive scrubbing agents that may scratch the surfaces. **Prior to cleaning entire surface, test cleaning agents on an inconspicuous area. Note that certain cleaning agents can be harmful to the surfaces of the product.** Periodically check all fasteners to make sure they remain secure.

WARRANTY

The manufacturer warrants products to be free of defects in materials, workmanship or design under normal use for five years on all products, from the date of purchase. This warranty does not include damage resulting from accident, abuse or misuse, alteration, or negligence of the product and expressly excludes normal wear of parts (including but not limited to vinyl). Contact us for full warranty details.



www.fab-ent.com