




## CLASSIC SERIES TREATMENT TABLES

with ADJUSTABLE BACKREST  
& with NON-ADJUSTABLE FLAT TOP

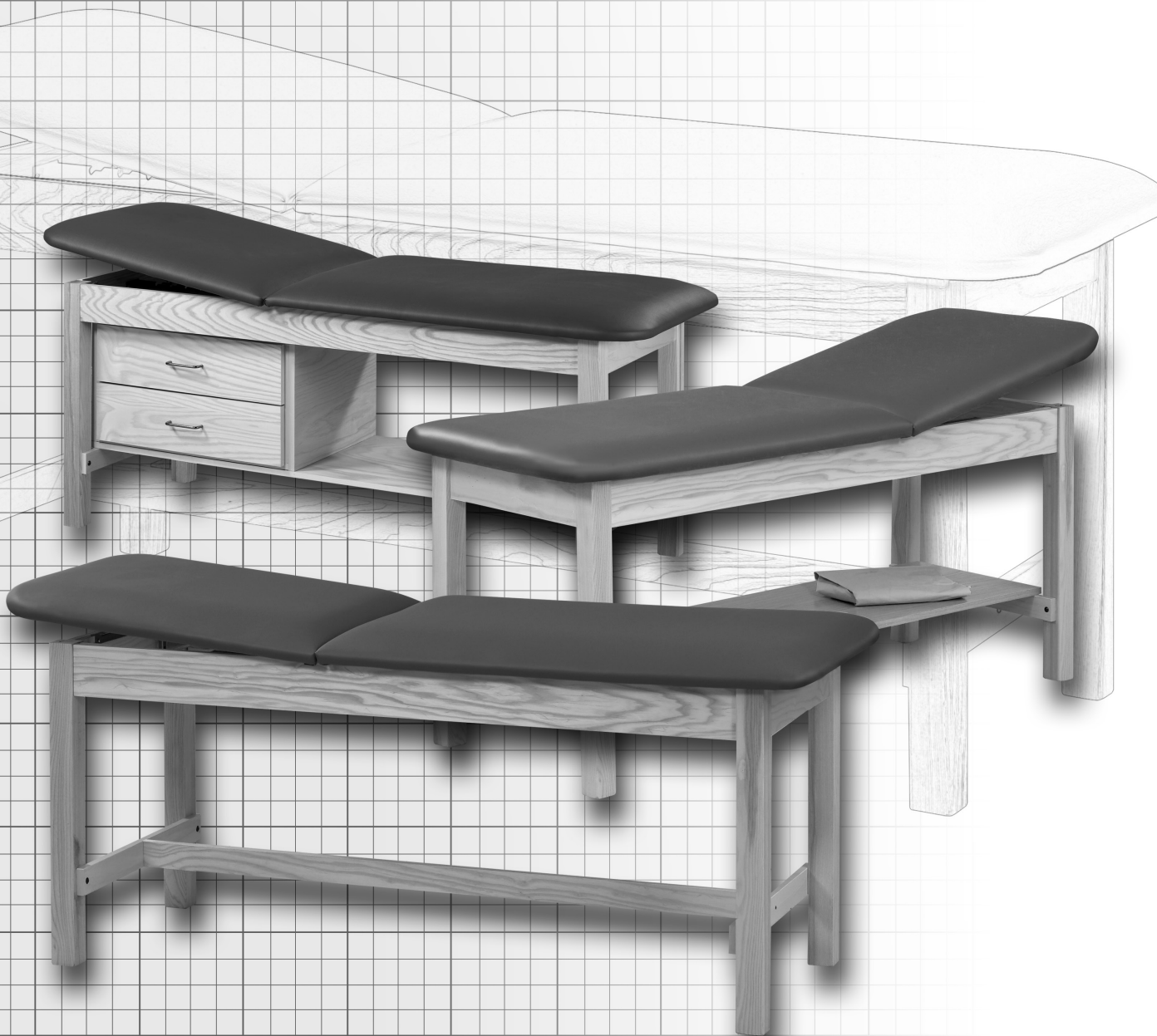

ASSEMBLY INSTRUCTIONS

# ASSEMBLY INSTRUCTIONS FOR CLASSIC SERIES TREATMENT TABLES

## MODELS with ADJUSTABLE BACKREST/KD & Tables with pre-attached backrest

**For MODELS with NON-ADJUSTABLE FLAT TOP & PRE-ATTACHED TOPS begin assembly with STEP 4**

### MAXIMUM WEIGHT CAPACITY 400 LBS

Thank you for counting on us to meet your medical equipment needs by purchasing this fine equipment. All equipment is designed and manufactured to provide years of dependable service because we want you as a customer now and in the future. Below are easy to follow instructions on how to properly assemble your equipment. Please contact us if you experience any difficulty with the assembly or have questions.

#### INTENDED USE

Treatment tables are specifically designed and engineered for the use by a single person at a time, who is attended to and observed by trained medical personnel. Never leave anyone on the table unattended. Person's weight and the combination of force exerted on the table during treatment is not to exceed a total of 400 lbs.

KD Quantity	Hardware Package	Pre-Attached Quantity
(4)	1 1/2" Truss Head Screws	(4)
(8)	1" Truss Head Screws	(0)
(8)	5/16" Washers	(4)
(4)	5/16" Nuts	(4)
(2)	2 1/4" Pan Head Screws	(2)
(2)	3/4" Barrel Bolts	(2)
(4)	1 1/4" Truss Head Screws (1030 & 1013)	(4)

#### Tools Required

Phillips Screwdriver and Adjustable Wrench

#### IF TOP IS ATTACHED TO FRAME, SKIP TO STEP 4

1

Place the upholstered sections on a flat surface with the foot end on the left and fabric tag toward the center of the table as shown. Place the head end on right with hinge toward the center of the table. Place frame on top with channel, teeth down, above the head end.

#### FOR MODELS WITH ADJUSTABLE BACKREST



fabric tag

center hinge

teeth

2

Align the holes in the hinge with the holes in the center board and fasten to center board using 4 silver, 1" truss head screws.

#### FOR MODELS WITH ADJUSTABLE BACKREST



3

Finger tighten the top's foot end to frame using 4 silver, 1" truss head screws and 5/16" washers.

Slide the foot end toward the center board and

finish tightening screws.

#### FOR MODELS WITH ADJUSTABLE BACKREST



4

Position leg in the leg brace with H- brace hole facing inward. Attach the leg using  $\frac{5}{16}$ " washer and  $\frac{5}{16}$ " nut. Finger tighten nut now, and fully tighten after step 6 is completed. Repeat with remaining legs.



5

Position the H-braces with finished edge toward top with the cut-out slot facing toward the inside of the table and attach using the four (4)  $1\frac{1}{2}$ " truss head screws.



6

Insert the barrel bolts into the holes at each end of the spanner. Make sure the threaded hole in the bolt is position to accept the pan head screw that will be inserted through the H-brace.




7

Insert the  $2\frac{1}{4}$ " pan head bolts through each H-brace and into the barrel bolts in the spanner. Finger tighten until the screw catches the threads in the barrel bolt, then secure with a screwdriver.



8


Return to each corner leg and finish tightening each nut. **DO NOT OVER TIGHTEN.**

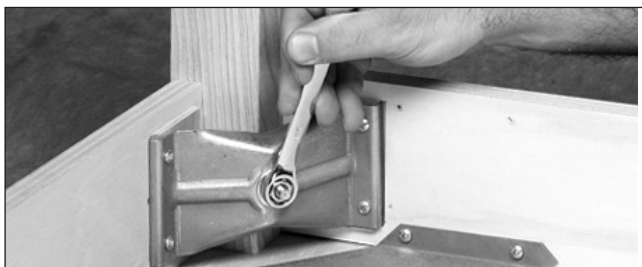

9

#### 1020 SHELF ASSEMBLY

Invert the table to an upright position on all four legs, using caution when inverting the end with the adjustable headrest. Apply 4 adhesive backed hook strips to top edge of H-braces. Place the full-length shelf on the spanner with the loop strip underside over the hook strips on the braces.

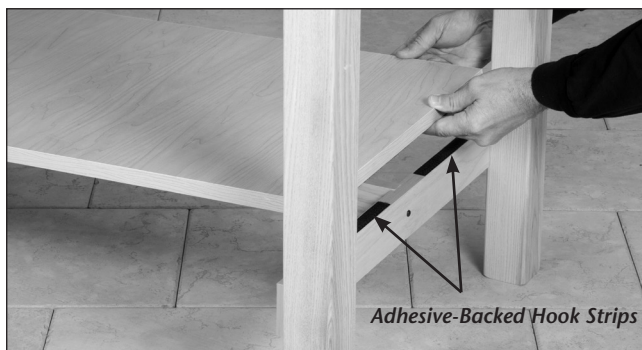

**FOR SAFETY PURPOSES, TAKE TIME TO CHECK ALL SCREWS, BOLTS AND NUTS TO INSURE THAT THEY ARE SECURELY TIGHTENED AND THAT THE TABLE HAS BEEN PROPERLY ASSEMBLED ACCORDING TO THESE INSTRUCTIONS.**



# 10

## SHELF/DRAWER ASSEMBLY


Using assistance, place the drawer box (with the drawers removed) or tier shelf on the full shelf, aligning it with the end of the shelf that has the pre-drilled holes.



# 11

## SHELF/DRAWER ASSEMBLY

Align pre-drilled holes and edges of the drawer or tier shelf section. Fasten shelf by screwing the supplied 1 1/4" black truss head screws through shelf and into pre-inserted, threaded inserts.

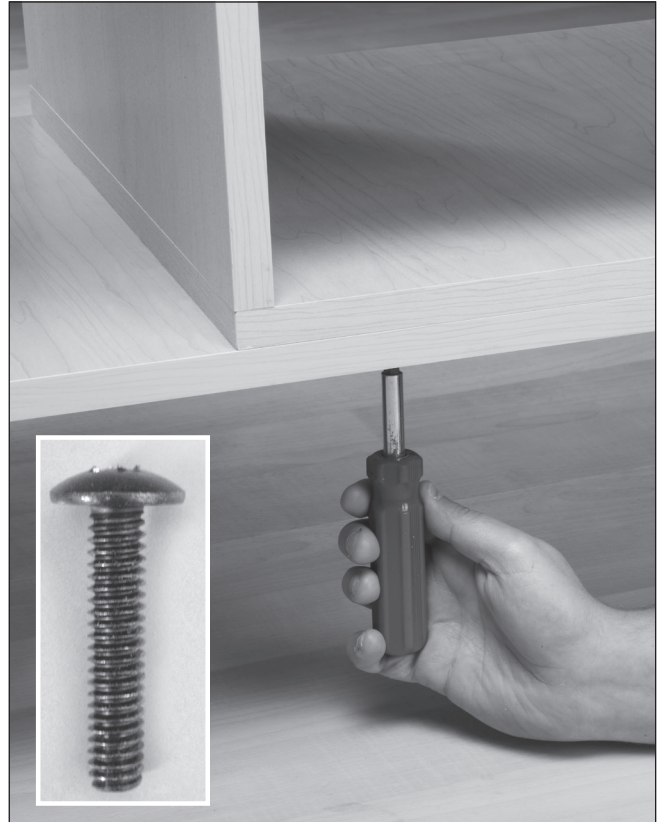

## Care & Cleaning

These products are built to be durable. With proper care, your equipment will provide years of superior looks and performance. Clean table with a soft cloth and any mild cleaning agent that may be required for sanitation. NEVER use abrasive scrubbing agents that may scratch the surfaces. **Before using any cleaning agents, try them on a small inconspicuous area to insure that they will not harm the upholstery or any other surface.** Damage from cleaning is not covered under the warranty. **Do not use alcohol based cleaning agents.** Periodically check that all fasteners are secure and operating properly.