



# ASSEMBLY INSTRUCTIONS

## PLATFORM MOUNTED PARALLEL BARS




# ASSEMBLY INSTRUCTIONS/PLATFORM-MOUNTED PARALLEL BARS

MAXIMUM WEIGHT CAPACITY 400 LBS

Thank you for counting on CanDo® to meet your physical therapy needs by purchasing this equipment. All of our equipment is designed and manufactured to provide years of dependable service because we want you as a customer now and in the future. Below are easy to follow instructions on how to properly assemble your equipment. Please contact us, if you experience any difficulty with the assembly or have questions.

## INTENDED USE

CanDo® Parallel Bars are specifically designed and engineered for physical therapy use only. Equipment is intended to be used by a single person at a time, who is attended to and observed by trained medical/physical therapy personnel. Never leave anyone use the parallel bars unattended. Person's weight and the combination of force exerted on the equipment during treatment is not to exceed the weight capacity for that specific model.

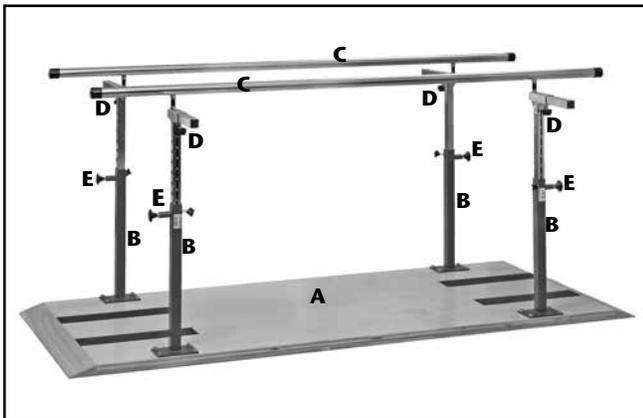

- A. Platform with Safety Strips
- B. 4 Telescoping Upright Posts with Number Indicator Strips
- C. 2 Stainless Steel Handrails
- D. Width Adjustment
- E. Height Adjustment

## Hardware

- (16) 1 1/2" Hex Bolts
- (16) 7/8" Washers
- (4) Small Star Knobs
- (4) Width Adjustment Star Knobs

## Tools Required



- Socket Wrench

## Time Required

- 20 – 30 Minutes


1

Mount all four telescoping posts to the platform, using 1 1/2" Hex Bolts and 1" washers on each post. Secure tight with a socket wrench.

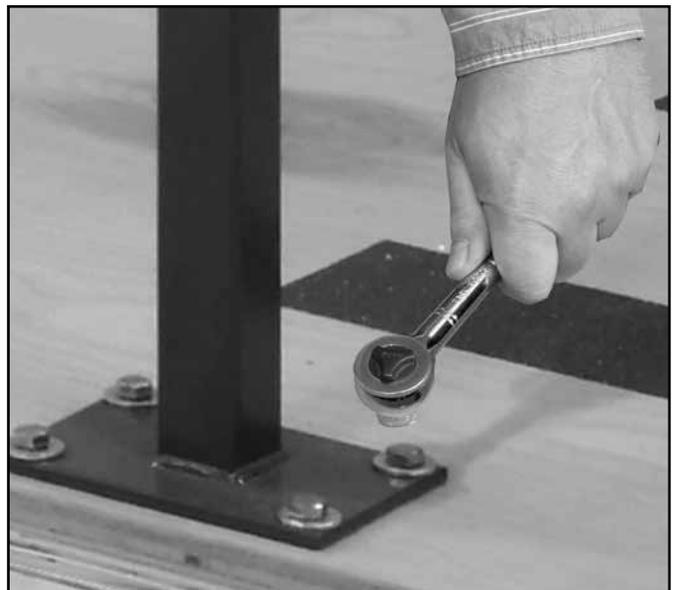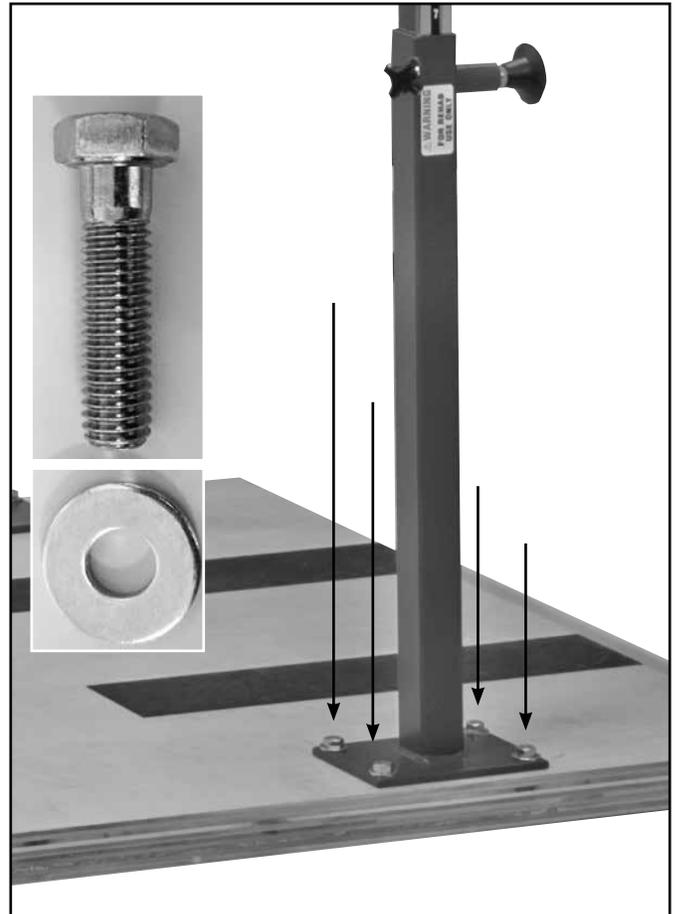


2

Slide the width adjustment bar that is attached to the handrails through the sleeve on top of each upright post. Adjust to desired width and tighten the width adjustment star knob to secure in place. Repeat the process for each upright.

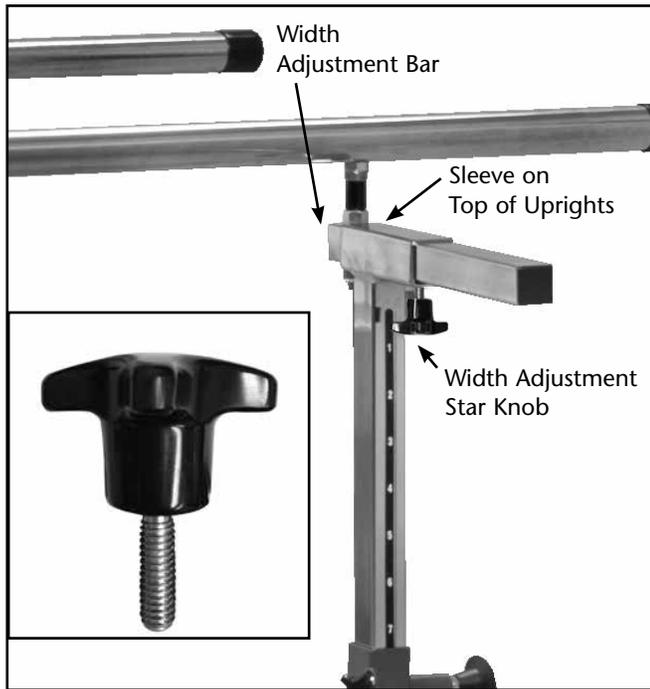

3

Adjust the height of each handrail with the spring-loaded plunger on each upright. Height is selected with the help of the number indicator strip. After adjusting the plunger, tighten the small star knob next to the plunger on the upright. Repeat the process for all uprights.

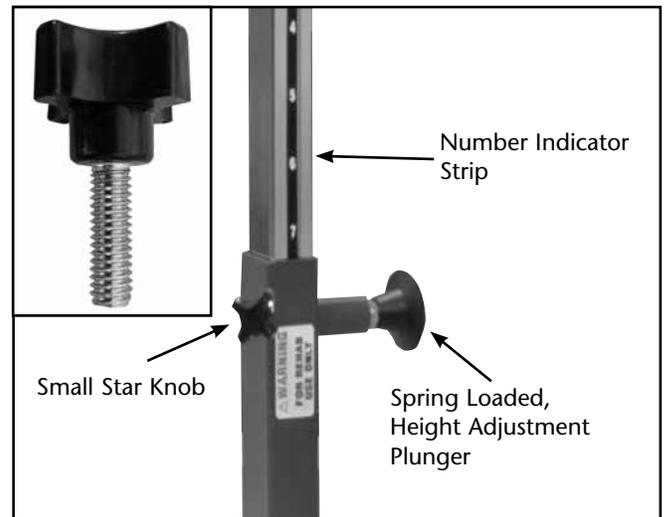

4

Clean with disinfectant according to Cleaning Guidelines and facility standards prior to use.

**FOR SAFETY PURPOSES, TAKE TIME TO CHECK ALL FASTENERS AND STAR KNOBS TO INSURE THAT THEY ARE SECURELY TIGHTENED AND THAT THE EQUIPMENT HAS BEEN PROPERLY ASSEMBLED ACCORDING TO THESE INSTRUCTIONS.**

## CARE & CLEANING

CanDo® products are built to be durable. With proper care, your equipment will provide years of superior looks and performance. Clean table with a soft cloth and any mild cleaning agent that may be required for sanitation. NEVER use abrasive scrubbing agents that may scratch the surfaces. **Prior to cleaning entire surface, test cleaning agents on an inconspicuous area. Note that certain cleaning agents can be harmful to the surfaces of the product. Periodically check all fasteners to make sure they remain secure.**

## WARRANTY

The manufacturer warrants products to be free of defects in materials, workmanship or design under normal use for five years on all products, from the date of purchase. This warranty does not include damage resulting from accident, abuse or misuse, alteration, or negligence of the product and expressly excludes normal wear of parts (including but not limited to vinyl). Contact us for full warranty details.



[www.fab-ent.com](http://www.fab-ent.com)