

PO BOX 905 VALLEY CITY, ND 58072 701/845-3984

1-800-437-8011

OPERATION AND MAINTENANCE MANUAL MODEL TW6121.SD, TW6121.BB and TW6121.NB/TG2724 POWER LIFT

READ THIS MANUAL PRIOR TO OPERATION
**LIMITED WARRANTY REGISTRATION CARD ENCLOSED

NOTICE:

Design changes may have occurred in the product since this manual was published. The technical information found within this manual was correct at the time this manual was approved for publication.

TRI W-G, INC. reserves the right to make changes without prior notification, in design specifications or other improvements on our products without incurring obligations to add them to any device in existence.

TRI W-G, INC. does not assume any risk or liability for attachments, or their effects on this device, if the attachments are not manufactured, sold by and expressly approved by TRI W-G, INC.

When replacement parts are required, use only those replacement parts authorized by TRI W-G, INC.

If any information contained in this publication is not understood, the user should contact TRI W-G, INC. for assistance.

To The Owner/Operator

DO NOT use or operate this device until this manual has been read and understood.

Foreword

This manual has been compiled to assist the owner and/or operator with the correct operation, service and routine preventive maintenance procedures of the Models TW6121.SD, TW6121.BB and TW6121.NB.

In order to get the optimum performance and efficiency that has been designed into the device, read this manual thoroughly before operating or servicing the device. Become familiar with all decals and safety messages within the manual and on the device. Keep this manual in a convenient place for easy reference when problems arise. DO NOT attempt to make repairs or adjustments you do not understand. If you require additional information or service, contact TRI W-G or your TRI W-G representative.

The signal words, CAUTION, WARNING or DANGER are used to indicate degree of hazards and to warn against unsafe practices that may cause personal injury and are used with appropriate safety instructions. These signal words are not to be ignored, you and/or your patients safety is involved.



CAUTION: Denotes a general reminder of good safety practices or directs attention to unsafe practices.



WARNING: Denotes a hazard intermediate between DANGER and CAUTION.



DANGER: Denotes the most serious hazard.

MODELS TW6121.SD, TW6121.BB AND TW6121.NB TABLE OF CONTENTS

| I/15-97 | PAGE 1 | | | |
|---|---|---|-----|------|
| SERVICE MAINTENANCE LOG | | | | 11 |
| SERVICE MAINTENANCE PROGRAM | | | | 10 |
| SERVICE PARTS BREAKDOWN MODEL TW6121.SD | | | | |
| SERVICE | • | • | | 7 |
| SUGGESTED CLEANING PROCEDURES FO | R VINYL | | | 7 |
| SPECIFICATIONS | | • | | 6 |
| OPERATION OF TABLE | | | | 5 |
| TABLE SET-UP | | | | 5 |
| SAFETY SUGGESTIONS | | | | 4 |
| GENERAL DESCRIPTION | | | | 3 |
| INTRODUCTION | | | | 3 |
| USER RESPONSIBILITY/TRACEABILITY | • | | Ins | sert |
| WARRANTY | • | | | 2 |
| TABLE OF CONTENTS | | | | 1 |
| FOREWORD | | | | ii |
| NOTICE | | | | i |

LIMITED WARRANTY

Such warranties are extended only with respect to the Purchase of the Product or Part as new merchandise directly from TRI W-G, INC. or a TRI W-G, INC. This Product or Part is sold by TRI W-G, INC. under the limited warranties set forth in the following paragraphs. futhorized Dealer and are extended to the ultimate customer purchaser of the TRI W-G, INC. Product or Part.

TRI W-G, INC. warrants that each Product or Part sold hereunder shall be free of defects in material and workmanship for the Product warranty period identified as follows:

Ten Years: structural frame.

Two Years: operating system and major moving components.

One Year: hand and foot controls; upholstery.

(Please note the aforementioned limited warranty is limited to original owner and not transferable.)

in the case of Products and twelve months in the case of Parts from the date of delivery from TRI W-G, INC. or an authorized TRI W-G, INC. Dealer, whichever is later. Should defects appear in any Products subject to this limited warranty and Parts subject to this limited warranty sold hereunder within the respective limited warranty period, TRI W-G, INC. will repair or replace under the terms of this limited warranty any defective Part or Parts or provide new or remanufactured Parts when the defective Part or Parts are returned to TRI W-G, INC. facilities at Buyer's expense upon twenty (20) days prior written notice to TRI W-G, INC. Buyer will be charged for any replacement Parts when shipped to Buyer by TRI W-G, INC. When the defective Parts are returned to TRI W-G, INC. returned goods authorization, charges will be waived.

This limited warranty does not apply to any Products or Parts which have been damaged through misuse, negligence or accident (including shipping damage) on the part of Buyer or any third party. This limited warranty does not apply to any Product in which Parts other than replacement Parts or Parts approved by TRI W-G, INC. have been used if said Parts are or may be the cause of failure.

G, INC. THIS WARRANTY IS IN LIEU OF FOR ANY BREACH OF WARRANTY, LIABILITY OF TRI W-G, INC. IS HEREBY EXPRESSLY LIMITED, AND SUBJECT TO ALL OTHER WARRANTIES, EXPRESSED FOR ANY SPECIAL, INCIDENTAL OR CLAIM FOR BREACH OF WARRANTY FOR PRODUCTS OR PARTS FURNISHED AGAINST TRI W-G, INC. MORE THAN THE TERMS HEREIN, TO THE REPAIR OR PART WHICH SHALL APEAR TO BE DEFECTIVE IN THE OPINION OF TRI W-OR IMPLIED, INCLUDING WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. IN NO EVENT SHALL TRI W-G, INC. BE LIABLE ASSERTED OR REPLACEMENT OF ANY PRODUCT IMPLIED. CONSEQUENTIAL DAMAGES. O.R Ö HEREUNDER EXPIRATION EXPRESSED PERIOD.

TRI W-G, INC. 215 12TH AYENUE NE VALLEY CITY, ND 58072 1-800-437-9011 1-701-845-3984

INTRODUCTION

This manual has been developed to describe the operation and proper maintenance procedures for the Model TW6121.SD, TW6121.BB and TW6121.NB. Please read this manual thoroughly before operating table and follow all recommended precautionary measures.

GENERAL DESCRIPTION

The below listed models have been designed as 90 degree tilt tables allowing patient to easily step on to a footboard for tilting from a vertical to a horizontal position providing for exam or treatment. These tables (TW6121.NB and TW6121.BB) are designed with an electric actuator requiring 120 volt line current which is then converted to 24 volt DC through means of a converter. The electrical actuator then functions using 24 volt current. As an option, a factory installed converter with battery power pack (TW6121.BB) will allow for lowering the table top to a horizontal position during time of power outage. The battery becomes charged during normal usage. The number of cycles in which table top can be tilted without hook up to 120 volt current is limited based on age and usage of battery. These tables are designed for use by patients with Circulatory, Neurological and Musculoskeletal conditions. Features of these models are listed below:

MODEL TW6121.SD (STANDARD MODEL)

- *Basic flat top
- *Power lift (110/220 volt)
- *Removable footboard
- *90 degree tilt range
- *2" foam cushion
- *Safety restrains
- *350 lb. patient load

MODEL TW6121.BB (BATTERY BACKUP)

- *Basic flat top
- *Power lift (110 volt DC drive)
 24 volt actuator with battery backup
- *Removable footboard
- *90 degree tilt range
- *2" foam cushtion
- *Safety restrains
- *350 lb. patient load

MODEL TW6121.NB (NO BATTERY)

- *Basic flat top
- *Power lift (110 volt DC drive)
- 24 volt actuator with no battery backup
- *Removable footboard
- *90 degree tilt range
- *2" foam cushion
- *Safety restrains
- *350 lb. patient load

Whenever you see this symbol, it means Attention! Become Alert! You and/or Your Patients Safety is involved.

Operating instructions must be given to everyone before operating this device.

Safety

Essential Safety Precautions

SAFETY SUGGESTIONS

TRI W-G can not anticipate every possible circumstance that might involve potential hazard. The warnings, cautions and safety suggestions in this manual are therefore not inclusive. If an operating procedure, device, maintenance or work method not specifically recommended by TRI W-G is used, you must satisfy yourself that it is safe for you and others. You must also ensure that the product will not be damaged or made unsafe by the procedures you choose.



CAUTION: Denotes a general reminder of good safety practices or directs attention to unsafe practices.



WARNING: Denotes a hazard intermediate between DANGER and CAUTION.



DANGER: Denotes the most serious hazard.

Become familiar with the controls and their individual responses before operating table.

Become familiar with the following safety statements since they will be found throughout this manual.



CAUTION: To avoid personal injury, lock caster prior to use.



CAUTION: To avoid damage to electrical equipment, use proper power source.



WARNING: To avoid personal injury, assure patient is properly secured with restraining straps.



CAUTION: Be sure patient has a good grip on the table frame hand hold when lower/raising table and that patient's feet are firmly located on the footboard. Instruct patient to lean in to table when lower/raising table. Instruct patient not to dismount until table has come to complete stop.



WARNING: Do not attempt to service any electrical components. Let only a qualified technician do any electrical repair work.

TABLE SET-UP

- Remove table from shipping crate. Inspect for any concealed damage that may have occurred during shipment.
 To assure cushions do not become damaged while moving table through door ways, keep cardboard packing liner
 on top of cushions.
- 2) Visually check to assure all fasteners are in place and secure.

OPERATION OF TABLE

1) After table is properly located, lock both wheels by depressing locking mechanism with sole of foot.



CAUTION: To avoid personal injury, lock casters prior to use.

2) (120 V, 60 Hz power supply required) Ensure proper power supply and then plug power cord into a (three-prong) grounded hospital grade receptacle.



CAUTION: To avoid damage to electrical equipment, use proper power source.



WARNING: Do not attempt to service any electrical components. Let only a qualified technician do any electrical repair work.

- 3) To raise table when table top is in the horizontal position and footboard is in the unlocked position (dropped down):
 - 1) Raise footboard to the locked position (This means manually lifting up on the footboard so that it remains locked in the horizontal position). This is necessary so that the safety limit switch will allow raising of table top.
 - 2) To move the table top from the horizontal position to the vertical position, depress the UP button on the air foot control.
 - 3) To move the table top from the vertical position to the horizontal position, depress the DOWN button on the air foot control. The air control switch is of a momentary design in that the table top will only tilt when switch is depressed.
 - 4) To drop the footboard out of the way when table is in horizontal position, depress the push button on the bottom side of the footboard. When the footboard is dropped, the table top can not be tilted until the footboard has been raised and locked in the horizontal position.



CAUTION: Be sure patient has a good grip on the table frame hand hold whenlowering/raising table and that patient's feet are firmly located on the footboard. Instruct patient to lean in to table when lowering/raising table. Instruct patient not to dismount until table has come to complete stop.



WARNING: To avoid personal injury, assure patient is properly secured with restraining straps.

SPECIFICATIONS TILT EXAM/TREATMENT TABLE TW6121.SD, TW6121.BB and TW6121.NB (STANDARD MODEL), (BATTERY BACKUP) and (NO BATTERY)

LENGTH:

78"

WIDTH:

28"

HEIGHT:

33'

TILT RANGE:

0 to 90 degrees

FRAME:

CASTERS:

4" Polyurethane-wheels with roller bearings and dual locking brake.

TABLE TOP MATERIAL:

2" foam cushion over non-translucent sub-structure and steel frame.

BELTS, RESTRAINING:

One 8" wide belt with velcro closure

One 6" wide belt with velcro closure

One 6" wide belt with buckle closure

TABLE, COVER AND PAD:

FIRE RETARDANT**Vinyl cover over 2 inch polyfoam (or equivalent) pad. Many colors available, Easily cleanable with soap and water. Complies with California Bulletin 117.

FINISH:

Polyurethane platinum standard. Polyurethane parchment also available. Special colors available upon request.

ELECTRICAL:

TW6121.SD Tilt: electro-mechanical drive, 3.3 amps, ordinary 120 volt, single phase, 60 Hz, 3-wire grounded circuit hospital grade plug. Available 220 volt, 50 Hz.

TW6121.BB and TW6121.NB Tilt: 24 Volt DC electro-mechanical drive powered by converter requiring 120 Volt, single phase, 60 Hz current, equipped with 3 conductor hospital grade plug. Option: factory installed battery pack included in converter (TW6121.BB).

WARRANTY:

60 months in the case of Products and 12 months in the case of Parts from the date of delivery.

TABLE LOAD:

WEIGHT:

**Flammability requirements for materials are those of the manufacturer and not that of TRI W-G, Inc. This term and any corresponding data refer to typical performance in manufacturer's tests and should not be construed to imply the behavior of this or any other material under other fire conditions.

Every effort has been made to use accurate photography, drawings, product specifications and descriptions at the time of printing. However, because of continuous, progressive product improvement, all specifications and descriptions are subject to change without prior notice.

^{*}Nominal adjustments stated.

SUGGESTED CLEANING PROCEDURES FOR VINYL

- ORDINARY DIRT Ordinary dirt can be removed by washing with warm water and a mild soap. Apply soapy
 water to a large area and allow to soak for a few minutes. Brisk rubbing with a cloth should then remove most
 dirt. The procedure may be repeated in case of stubborn or imbedded dirt. A soft bristle brush may be used after
 the soap has been applied.
- 2) BALL POINT INK Ball point ink may sometimes be removed if rubbed immediately with a damp cloth using water or rubbing alcohol.

<u>NOTE</u> - Lysol (of any kind) should <u>NOT</u> be used on this type of material. The naugahyde turns white when Lysol comes in contact with it.

SERVICE

Any questions regarding operating or maintenance of table should be directed to the service department of TRI W-G, INC.

MAIL


TRI W-G, INC. 215 12th Avenue NE PO Box 905 Valley City, ND 58072-0905

TELEPHONE

In all states, except North Dakota 1-800-437-8011

In North Dakota 1-701-845-3984

SERVICE PARTS LIST MODEL TW6121.BB and TW6121.NB

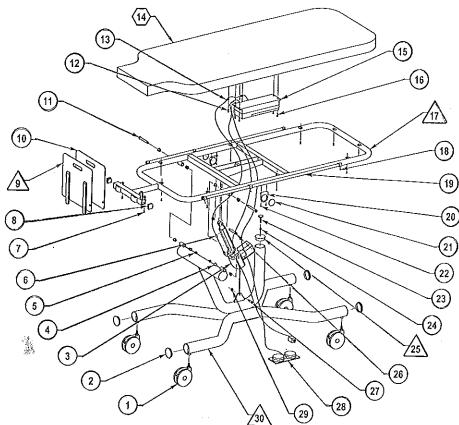

| | | 2307 | |
|----------|-------------|---|------|
| ITEM NO. | PART NUMBER | <u>DESCRIPTION</u> | QTY. |
| 1 | 903-001-049 | Casters w/brakes | 4 |
| 2 | 907-005-077 | Endcap, plastic | 4 |
| 3 | 900-010-048 | Magnetic matrix DC actuator | 1 |
| 4 | 907-003-012 | Plastic bushing | 8 |
| 5 | 905-044-004 | Nut, nylock | 2 |
| 6 | 900-011-003 | Self closing clip | 2 |
| 7 | 905-069-011 | Screw, pan head phillips | 11 |
| 8 | 907-005-076 | Square black plastic end cap | 2 |
| 9 | 621-013-000 | Footboard weldment | 1 |
| 10 | 621-014-000 | Footboard mat | 1 |
| 11 | 905-020-019 | Screw, shoulder, socket | 2 |
| 12 | 905-069-003 | Screw, round head phillips tapping | 2 |
| 13 | 900-012-006 | Loop clamp - black | 2 |
| 14 | 612-028-000 | Cushion | 1 |
| 15 | 900-010-053 | Power supply with battery (TW6121.BB only) | 1 |
| 15 | 900-010-055 | Power supply without battery (TW6121.NB only) | 1 |
| 16 | 905-069-002 | Screw, round head phillips tapping | 4 |
| 17 | 612-056-000 | Top frame weldment | 1 |
| 18 | 905-019-001 | Screw, socket set | 6 |
| 19 | 612-015-000 | Restrainer bar | 2 |
| 20 | 129-017-000 | Inclinometer bracket | 2 |
| 21 | 917-000-143 | Inclinometer | 2 |
| 22 | 905-074-004 | Screw, button head socket | 4 |
| 23 | 903-003-002 | Adjustable stop | i |
| 24 | 905-052-009 | Nut, hex jam | 1 |
| 25 | 621-012-000 | Adj. horizontal cap weldment | 1 |
| 26 | 905-091-004 | Screw, shoulder, socket | i |
| 27 | 905-091-003 | Screw, shoulder, socket | 1 |
| 28 | 900-005-048 | Foot control | 1 |
| 29 | 905-092-002 | Nut, nylock | 2 |
| 30 | 612-057-000 | Base frame weldment | 1 |
| | | PAGE 8 | • |
| | | | |

SERVICE MAINTENANCE PROGRAM

This device has been designed and built to provide many years of reliable service. To keep this device in good operating condition, a service maintenance program should be established to assure continued trouble-free service.

The below items should be included in the service maintenance program. A check list has been provided at the back of this manual to provide an area to document the date of inspection and comments.

- 1) Assure motor/actuator mount bolts are secure. NOTE: the pivot bolts are provided with a lock nut to provide movement of the actuator during operation.
- 2) Assure all other bolts and screws are tight.
- 3) Assure power cord does not show signs of wear or damage.
- 4) Visually inspect device while operating to assure it is functioning as it should.
- 5) Assure hand and/or foot control is functioning as it should.
- 6) Inspect restraining belt loop portion of the velcro strap to assure that it has not become weakened due to wear or breakage.

SIX (6) MONTH SERVICE MAINTENANCE LOG

| DATE OF INSPECTION | COMMENTS |
|--|---|
| | |
| · · · · · · · · · · · · · · · · · · · | |
| | |
| . Marient | |
| | |
| | |
| | |
| | |
| | |
| | |
| | |
| | |
| | |
| | |
| <u> </u> | |
| | |
| | |
| La de Principal III de la Companya d | Thirds. |
| | |
| | |
| | |
| | |
| | |
| A 100-10 11 | |
| | |
| | 111 11 11 11 11 11 11 11 11 11 11 11 11 |
| · | |
| | |
| | |
| | |
| | |
| | ************************************** |
| | |
| | |
| | |
| | |
| | |
| | |