



215 - 12th Avenue NE  
Valley City, ND 58072-0905

1-800-437-8011  
triwg@triwg.com  
www.triwg.com

#### FOR CUSTOMER ASSISTANCE

As a future reference, we suggest  
you record the information listed  
below for quick accessibility.

Model No.: \_\_\_\_\_

Serial No.: \_\_\_\_\_

Purchase Date: \_\_\_\_\_

Purchased From: \_\_\_\_\_

Tri W-G® Service:  
1-800-437-8011  
triwg@triwg.com

## **USER MANUAL**

*P/N: 920-002-031*

*For*

### **Motorized Tilt Table TG2726**



Read this manual before installing, operating, or  
maintaining this table. Failure to follow safety precautions  
and instructions could cause a system failure and result in  
serious injury, death or property damage.

LIMITED WARRANTY REGISTRATION CARD ENCLOSED



## **TABLE OF CONTENTS**



INTRODUCTION .....	2
1.0 RECEIVING INSPECTION .....	2
2.0 UNCRATING .....	2
3.0 INSTALLATION .....	2
4.0 SAFETY/PRECAUTIONARY INSTRUCTIONS .....	3
5.0 INTENDED USE .....	3
6.0 INTENDED USER PROFILE .....	4
7.0 OPERATING INSTRUCTIONS .....	4
8.0 MAINTENANCE .....	4
9.0 SYMBOLS, INDICATORS, and DEGREE OF SAFETY .....	6
10.0 WARNINGS and USAGE HINTS .....	7
11.0 OTHER HAZARDS .....	7
12.0 SPECIFICATIONS .....	8
13.0 STARTUP CONDITIONS .....	8
14.0 OPERATING CONDITIONS .....	8
15.0 STORAGE CONDITIONS .....	8
16.0 TRANSPORT CONDITIONS .....	9
17.0 NOTICE .....	9
18.0 TROUBLESHOOTING .....	9
19.0 REPLACEMENT PARTS .....	10
20.0 DISPOSAL .....	11
WARRANTY .....	11
DISCLAIMER .....	12

## **INTRODUCTION**

The purpose of the User Manual is to introduce the authorized user to the operating instructions on how to use and work with this medical device safely, and to conduct routine preventative maintenance procedures on the device. For this goal to be achieved, it is essential that all the authorized users/operators (“user”) read this manual carefully, and understand and practice the precautionary safety measures recommended in it. The owner and/or operating authority determines who the user is.

**ANYONE OPERATING THIS DEVICE MUST READ THIS USER MANUAL.**

### **1.0 RECEIVING INSPECTION**

1.1 Inspect for visible damage of the container. If there is outside damage, note on the shipping documents, and report to commercial carrier. Determine whether or not to accept or refuse the container. Sign if appropriate.

### **2.0 UNCRATING**

2.1 Upon acceptance of the container, inspect for any damage to the device immediately following uncrating. In the event the device is damaged, file a merchandise damage report immediately with the delivering carrier. **DO NOT** return damaged merchandise to Tri W-G® unless instructed to do so. Call for a Return Authorization Number prior to returning any merchandise.

2.2 Keep the cardboard liner on top of the table while moving tilt table to the desired location. This will prevent damage to the frame and/or cushion while moving the table down hallways, through doorways, etc.

2.3 Remove all other packaging material from the table, except for the cardboard cushion liner/protector.

2.4 Properly dispose of/or recycle all packaging material.

### **3.0 INSTALLATION**

3.1 **WARNING! Excessive Weight Hazard.** Use appropriate pallet jack(s), lifting devices, and/or equipment, adequate to lift approximately 650 lbs minimum, and we recommend four or more people to move table. Failure to do so may result in personal injury, death, and/or property damage. When moving the table to its desired location, use proper moving/lifting techniques and/or body mechanics per commercial carrier and/or the facility handling protocol or guidelines; this due to its immense weight and overall size.

3.2 There is no assembly required.

3.3 The tilt table should be placed in a location based on its intended use. Position the table in an area having flat/level floors so that the table is not distorted out of alignment. By following this instruction, the casters with locking brakes will function as intended to achieve maximum stability.

3.4 All four (4) of the total locking casters should have their wheels locked at all times except for moving. **DO NOT** move the table with a patient on the table.

3.5 Connect or plug-in the power cord to a properly grounded 120 Volt AC receptacle, conforming to the relevant state and nationally recognized electrical codes, and follow the procedure outlined in the Safety/Precautionary Instructions.

3.6 The power supply cord serves as a disconnect device or as the on/off switch. Do not unplug by pulling on the cord. To unplug, grasp the plug, not the cord. **DO NOT** handle plug with wet hands. The receptacle shall be installed near the table and shall be easily accessible.

3.7 Arrange power supply cord away from the traffic area and where it will not be tripped over.

3.8 **DO NOT** use with an extension cord.

3.9 Upon connecting the power supply cord to a grounded receptacle, test treatment table to determine electrical functionality.


3.10 **STAY ALERT**, watch what you are doing and use common sense.

#### **4.0 SAFETY/PRECAUTIONARY INSTRUCTIONS**

4.1 Once again, please have anyone who is authorized to use this table read, understand, and practice the safety and precautionary instructions recommended in this user manual. All operators should use common sense, know and fully understand the hazards and risks associated with using and/or operating this Tri W-G, Inc. table prior to operating it.

4.2 **NEVER LEAVE THE PATIENT UNATTENDED.** Close attention is necessary when patients are children and/or disadvantaged.

4.3 Never place ones hand or feet, nor the patients' hand or feet, near any of the working mechanisms or gap areas on the table when raising or lowering the table. **DO NOT** place or leave objects or items of any kind underneath the base frame of the table at any time.

4.4 Whenever you see this symbol:  it signifies "Caution", and is used as a "General Warning Sign"; You and/or Your Patients' Safety is Involved.

4.5 During table operation, immediately stop the table operation, if anything unusual is observed or unusual sounds are heard coming from the table.

4.6 **DO NOT** use this table to play on or as a play area, no matter what the age of the patient or individual may be. It is a Class I Medical Device, called a tilt table. And **DO NOT** store or allow any object, of any kind, under the table at any time. No medical claims are made with regard to a patient using the device. Tri W-G,<sup>®</sup> Inc. shall not be liable for any damage, death or injury caused, directly or indirectly, from the misuse of this device/table; and the user shall indemnify Tri W-G,<sup>®</sup> Inc. from and against all costs, damages and expenses (including legal expenses) arising from any such misuse.

#### **5.0 INTENDED USE**

It is a powered table intended for medical purposes (Class I Device) that is an electrically operated flat surface table that can be adjusted to various positions. It is used by patients with circulatory, neurological, or musculoskeletal conditions to increase tolerance to an upright or sitting position; [21CFR890.3760]. At no time should patients be left unattended on the table.

## **6.0 INTENDED USER PROFILE**

It is intended to be used by Licensed Physical Therapists, Assistant Physical Therapists, Occupational Therapists, Assistant Occupational Therapists, and licensed rehab specialists utilizing a prescribed therapeutic protocol, written by a licensed medical practitioner or physician. It is not intended to be used by maintenance staff, installers, patients and/or lay persons.

## **7.0 OPERATING INSTRUCTIONS**

7.1 **Adjust Height**--To raise or lower the table using the footswitch, use one's foot and press on the UP or DOWN button. Always check to make absolutely sure there are no objects under the table or near the top frame when lowering or raising the table. It takes very little time to make a second check if you have left the area prior to your next hi-lo adjustment. Never leave the table with a patient on it.

7.2 **Duty Cycle**--System is an intermittent operation; 1 minute on, 9 minutes off.

7.3 **Capacity**--The table is intended to support **500 lbs** (227 Kilograms), evenly distributed over the table top.

7.4 **Casters, Standard**--This standard caster is a total locking caster. It is a mobile and transportable tilt table, and may be moved from one location to another with the aid of the supporting twin wheel casters. No patient(s) and/or individuals whatsoever are allowed on the table during transportation of the table. The casters are only used for the purpose of moving the table from one place to another. Once the table is positioned on a level area, in its dedicated working place, totally lock all four casters. Activating the total locking caster is accomplished by placing foot on the trailing end of the caster housing and step down on the locking mechanism paddle. To unlock the casters, place foot on the front raised portion of the caster housing, locking step pad and push down. **DO NOT** position the tilt table on an uneven floor area that may twist/distort the table frame; as it may put the lifting mechanism in a stressed state, shortening the lifespan of the table. **ALWAYS** lock all four (4) casters during use of the tilt table, except for moving.


7.5 **User--Never** allow the patient to operate the tilt table. Only trained personnel authorized to use the table should operate the treatment table.

## **8.0 MAINTENANCE**

8.1 The table is equipped with a linear actuator system as follows: The linear actuator MAX3™ (Tri W-G® PN 900-010-055) is designed for a life of approximately 20,000 cycles (with authorized usage). The KOM Control Unit is only suitable for inside applications and must not be subjected to weathering strong UV radiation or corrosive or explosive air. However, conducting routine preventative maintenance checkups are a must. One must routinely check the actuator for wear on the end mounts and connecting bolts, for unusual noises, for actuator wobbling when extending or retracting the spindle, and that it is operating smoothly, as minimums. Due to normal wear, it may be necessary to replace the linear actuator during the lifespan of the table. The actuator is not-servicable in the field. Call customer service for a replacement actuator.

8.2 Once a month, and/or as may be needed, check the actuator spindle end for a gap that looks like the one shown in photo 8.2.1 below. If this gap looks closed or like it has been crushed or squeezed, please call the Tri W-G,® Inc. Service Department at 1-800-437-8011.

### 8.2.1

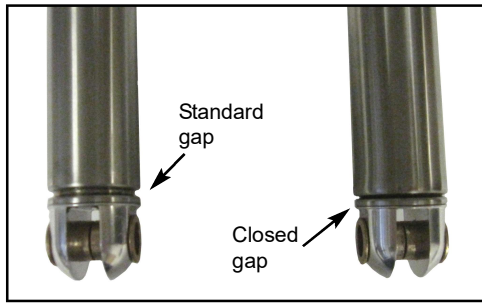

8.3 Due to normal wear, it may be necessary to replace the footswitch or the power cord. Call our customer service for replacement parts and/or installation instructions.

8.4 Cleaning the vinyl cover is accomplished with water and a mild detergent. Do not use Peroxide, Hylex, Lysol, strong or aggressive cleaning agents as they will discolor and reduce the life of the vinyl. The water used for cleaning, including chemical additives, must be pH-neutral. And liquids must not touch the actuator(s) during retraction or extension. A soft dust cloth will work to remove dust and lint on the steel frame when necessary. The mat table frame should be wiped down using a dry cloth to remove dust/lint on a regularly scheduled basis--keep free of dust. Cleaning the cover should be done after each patient procedure is complete and prior to new patient being placed on the table.

8.5 Inspection and maintenance/repair shall be made on a regular and/or periodic basis, which shall encompass the entire table. This includes, but is not limited to, the following: all moving parts, casters, actuators, backrest and knee gatch (if applicable), cushion (vinyl, velcro, stitching, foam), broken welds and/or cracks, bushings, nuts, bolts, screws, handset, power cord, wire holding clips and snaps, etc., to name a few. Replacement of worn or failed parts shall be replaced immediately, or as is deemed necessary. All inspections and maintenance/repair activities shall be recorded by owner facility. Operator should notify all users to discontinue using the table if the safety of operator and/or patient is eminent. Call Tri W-G,<sup>®</sup> Inc. Service Dept at 1-800-437-8011 and report any potential safety concerns.

8.6 Any repair should be performed by trained or certified service personnel.

## 9.0 **SYMBOLS, INDICATORS, and DEGREE OF SAFETY**



Mandatory action sign; Read User Manual Caution: General Warning Sign.



Mandatory action sign; Read User Manual Caution: General Warning Sign.



Read User Manual; contains useful information.



General Warning Sign.



Type B Equipment: Equipment providing a particular degree of protection against electric shock, particularly regarding allowable leakage current and reliability of the protective earth connection.



Protective Earth Ground: This product is Class I equipment in which protection against electric shock does not rely on BASIC INSULATION only, but which includes an additional safety precaution in that means are provided for the connection of the EQUIPMENT to the protective earth conductor in the fixed wiring of the insulation in such a way the ACCESSIBLE METAL PARTS cannot become LIVE in the event of a failure of the BASIC INSULATION.



Earth (ground).

WATER

Protection against harmful ingress of water--Prevent mat table from being subjected to water sprays or water hosing.

ANESTHETICS

Not suitable for use in the presence of a flammable anesthetic mixture with air or oxygen or nitrous oxide.

ELEC. RATING


Input: 120 VAC System--120 VAC; 1.6 A Max; 60 Hz  
Output: 24 VDC; 6A



## 10.0 WARNINGS and USAGE HINTS






Please make note of the meaning of the following warnings and usage hints:

**Note:** indicates usage information that helps the user to use the product correctly and efficiently or to understand the properties of the product.

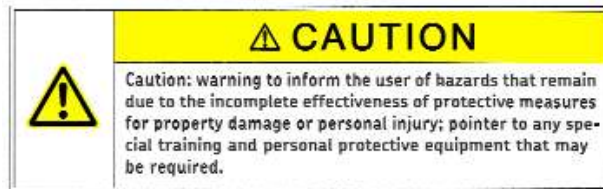

## 11.0 OTHER HAZARDS

The manufacturer has constructively, and with protective measures, minimized the effects of existing hazards. Pay attention to the following hazards and the possible countermeasures given in the following:

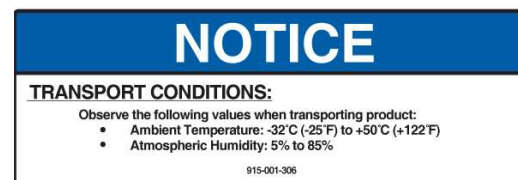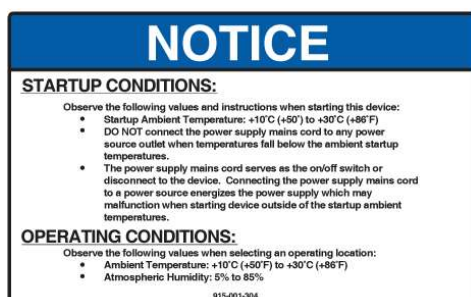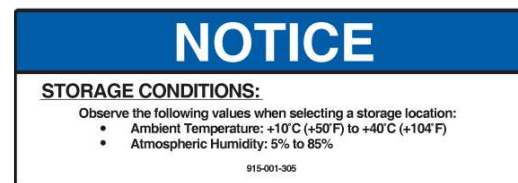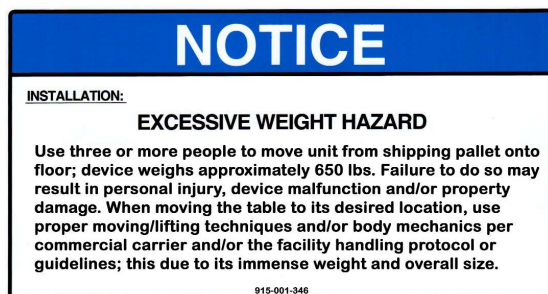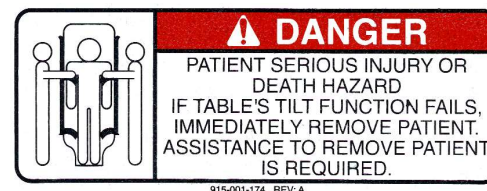

## **12.0 SPECIFICATIONS:**

Item No. TG2726: Width-31"; Length-81.5"; Height- 34.5" to top of cushion (*heights nominal*)

Tilt Range: 0 to 90 degrees; angle indicator

Lifting Weight Capacity: A maximum static uniform load of 500 lbs.

Upholstered Top: 2" polyfoam with vinyl cover standard; Vinyl Coated Nylon & Herculite, optional. Vinyl conforms to California Fire Code 117.

Restraining Belts: Belts included, set of 2

Casters: Four Swivel Casters with total lock mechanism, 5" diameter wheels, are standard.

Frame: Heavy gauge steel.

Finish: Polyurethane Pearl Grey standard; Porcelain optional.

Switch: Footswitch for adjusting to various table positions. Power supply serves as main disconnect device.

Electrical: Input: 120 VAC System--120 VAC; 1.6 A Max; 60 Hz; Hospital Grade Plug  
Output: 24 VDC; 6A

Approval: UL<sup>®</sup> Recognized Components  
UL 60601-1  
EN 60601-1  
ANSI/AAMI 60601-1

## **13.0 STARTUP CONDITIONS:**

- 13.1 Observe the following values and instructions when starting this device:
- Startup Ambient Temperatures are: +10° C (+50° F) to +30° C (+86° F)
  - **DO NOT** connect the power supply mains cord to any power source outlet when temperatures fall below the ambient startup temperatures.
  - The power supply mains cord serves as the on/off switch or disconnect to the device. Connecting the power supply mains cord to a power source energizes the control unit which may malfunction when starting device outside of the startup ambient temperatures.

## **14.0 OPERATING CONDITIONS:**

- 14.1 Observe the following values when selecting a operating location:
- Ambient Temperature: +10° C (+50° F) to +30° C (+86° F)
  - Atmospheric Humidity: 5% to 85%

## **15.0 STORAGE CONDITIONS:**

- 15.1 Observe the following values when selecting a storage location:
- Ambient Temperature: +10° C (+50° F) to +40° C (+104° F)
  - Atmospheric Humidity: 5% to 85%

## **16.0 TRANSPORT CONDITIONS:**

- 16.1 Observe the following values when transporting product:
- Ambient Temperature: -32° C (-25° F) to +50° C (+122° F)
  - Atmospheric Humidity: 5% to 85%

## **17.0 NOTICE:**

17.1 Design changes may have occurred in the product since this user manual was published. The technical information found within this manual was correct at the time this user manual was approved for publication.

17.2 Tri W-G,<sup>®</sup> Inc. does not assume any risk or liability for attachments, or their effects on this device, if the attachments are not manufactured, sold by, and expressly approved by Tri W-G,<sup>®</sup> Inc.

17.3 When parts are required, use only those parts authorized by Tri W-G,<sup>®</sup> Inc.

17.4 If any information contained in this publication is not understood, the user should contact Tri W-G,<sup>®</sup> Inc. for assistance at 1-800-437-8011.

17.5 Do Not perform any electrical or general service on the table prior to disconnecting the power supply cord which serves as the on/off switch to the table.

## **18.0 TROUBLE SHOOTING**

18.1 Table does not move up or down:

18.1.1 Check power cord connection to main power source.

18.1.2 Make sure main power source is receiving power.


18.1.3 Check handset, making sure it is plugged in.

18.1.4 Thermal protector may have activated, and you must allow the system to cool down. Duty Cycle is on/off intermittently 1 min/9 min.


18.2 Tried everything and nothing works to resolve the dilemma--contact Tri W-G,<sup>®</sup> Inc. Customer Service by calling 1.800.437.8011.

## 19.0 **REPLACEMENT PARTS**


These parts can be ordered from the Tri W-G,<sup>®</sup> Inc. Service Dept by calling 1-800-437-8011.

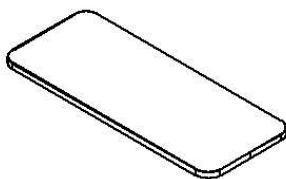


613-015-000  
ONE SECTION  
CUSHION 31"

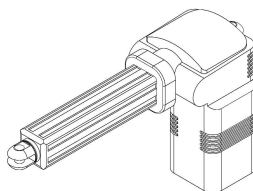


900-010-048  
ACTUATOR  
MAX3

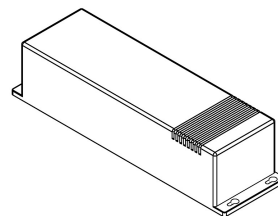


900-010-055  
CONTROL UNIT  
120 VOLT

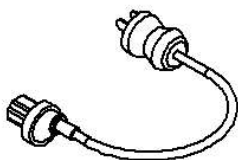

900-001-059  
POWER CORD  
120 VOLT

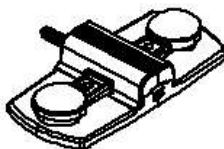

900-005-055  
FOOTSWITCH,  
ONE CHANNEL

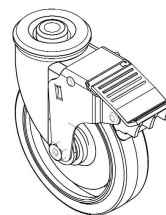

903-001-067  
CASTER

## **20.0 DISPOSING OF THIS MEDICAL DEVICE**

Deterioration in function, failure and/or the medical device has reached its life span, as a result of aging, wear, repeated use, or it has been determined to decommission it and/or dispose of it, must be done in a technically proper manner and in accordance with international, national, regional and/or local environmental regulatory requirements for disposing of medical devices. The owner of this medical device is responsible for its proper disposal.

### **LIMITED WARRANTY**

This Product or Part is sold by Tri W-G,<sup>®</sup> Inc. under the limited warranties set forth in the following paragraphs. Such warranties are extended only with respect to the Purchase of the Product or Part as new merchandise directly from Tri W-G,<sup>®</sup> Inc. or a Tri W-G,<sup>®</sup> Inc. Authorized Dealer and are extended to the ultimate customer purchaser of the Tri W-G,<sup>®</sup> Inc. Product or Part.

Tri W-G,<sup>®</sup> Inc. warrants that each Product or Part sold hereunder shall be free of defects in material and workmanship for the Product warranty period identified as follows:

Ten Years: structural frame.

Two Years: operating system

One Year: hand and foot controls; upholstery.

*(Please note the aforementioned limited warranty is limited to original owner and not transferable.)*

in the case of Products and twelve months in the case of Parts from the date of delivery from Tri W-G,<sup>®</sup> Inc. or an authorized Tri W-G,<sup>®</sup> Inc. Dealer, whichever is later. Should defects appear in any products subject to this limited warranty and Parts subject to this limited warranty sold hereunder within the respective limited warranty period, Tri W-G,<sup>®</sup> Inc. will repair or replace under the terms of this limited warranty any defective Part or Parts or provide new or remanufactured Parts when the defective Part or Parts are returned to Tri W-G,<sup>®</sup> Inc. facilities at Buyer's expense upon (20) days prior written notice to Tri W-G,<sup>®</sup> Inc. Buyer will be charged for any replacement Parts when shipped to Buyer by Tri W-G,<sup>®</sup> Inc. When the defective Parts are returned to Tri W-G,<sup>®</sup> Inc. pursuant to Tri W-G,<sup>®</sup> Inc. returned goods authorization, charges will be waived.

This limited warranty does not apply to any Products or Parts which have been damaged through misuse, negligence or accident (including shipping damage) on the part of Buyer or any third party. This limited warranty does not apply to any Product in which Parts other than replacement Parts or Parts approved by Tri W-G,<sup>®</sup> Inc. have been used if said Parts are or may be the cause of failure.

FOR ANY BREACH OF WARRANTY, EXPRESSED OR IMPLIED, THE LIABILITY OF TRI W-G,<sup>®</sup> INC. IS HEREBY EXPRESSLY LIMITED, AND SUBJECT TO THE TERMS HEREIN, TO THE REPAIR OR REPLACEMENT OF ANY PRODUCT OR PART WHICH SHALL APPEAR TO BE DEFECTIVE IN THE OPINION OF TRI W-G,<sup>®</sup> INC. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. IN NO EVENT SHALL TRI W-G,<sup>®</sup> INC. BE LIABLE FOR ANY SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES. NO CLAIM FOR BREACH OF WARRANTY FOR PRODUCTS OR PARTS FURNISHED HEREUNDER SHALL BE ASSERTED AGAINST TRI W-G,<sup>®</sup> INC. MORE THAN TWENTY (20) DAYS AFTER THE EXPIRATION OF THE WARRANTY PERIOD.

## **DISCLAIMER**

Every effort has been made to use accurate photography, drawings, product specifications and descriptions at the time of printing.

The contents of this publication are the copyright of the publisher and may not be reproduced unless prior written permission is granted. Every effort has been made to ensure the accuracy of the information contained in this Operator's Manual, but no liability can be accepted for any loss or damage whether direct, indirect, or consequential arising out of the use of the information contained herein. The label(s) on any of the products/components supercedes the copy and/or data that is in this User Manual. Copy, data and/or print on serial tags and labels attached to any of Tri W-G products and/or components supplied by other vendors used in Tri W-G products, shall supersede, any and all copy, data and/or print stated in this User Manual. However, because of continuous progressive product improvements, all specifications and descriptions are subject to change without prior notice.

Tri W-G® is a registered trademark of Tri W-G, Inc. UL® is a registered trademark of Underwriters Laboratories, Inc. All other company and product names are trademarks or registered trademarks of their respective owners.

*(This page intentionally left blank)*

Manufactured and Distributed by:



**Phone: 1-800-437-8011**

**Email: [triwg@triwg.com](mailto:triwg@triwg.com)**

215 12th Avenue NE

PO Box 905

Valley City, ND 58072-0905