Indications

- > Casted fracture
- > Bulky bandages

Contraindications

- > Open fractures

Cleaning Instructions/Maintenance

- > When the shoe is slightly dirty, clean it using a damp cloth or a soft brush.
- > Allow the shoe to air dry only.

Instructions for use

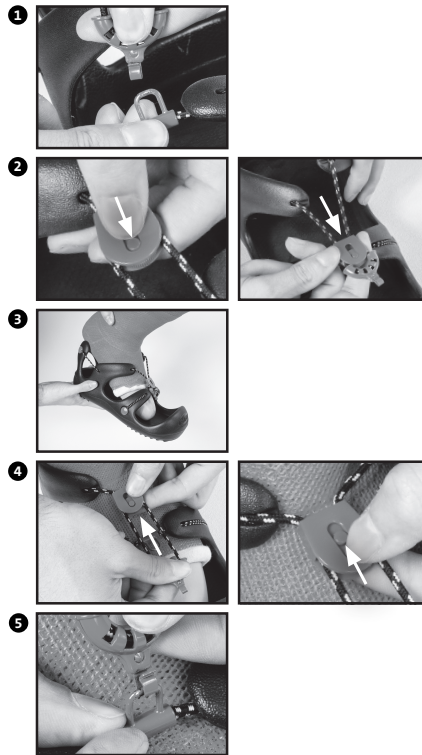
1. Unhook the bungee-closure system.
2. Press the fastener of the locking mechanism toward the toes in order to widen the ankle strap.
3. Place foot in cast shoe.
4. While holding the end of the bungee-closure system, move the locking mechanism towards the ankle and press the fastener up in the locking position.
5. Hook the bungee-closure system.

Caution

Please always follow these instructions for use. In case of pain or swelling, discontinue use and consult your medical provider immediately. This device is designed to assist recovery and should be used exclusively under the instruction of and in consultation with a healthcare professional. DARCO® International Inc. does not assume any responsibility for misuse of the product. Do not drive a vehicle while you are wearing this device. Use extreme caution when walking on all surfaces when using this device. Walk more slowly than normal. This device is for single-patient use only.

To the patient/user: This device has been designed and manufactured by DARCO® International, Inc. to be state of the art for treatment of a specific medical condition. Should any serious incident occur in relation to this device it should be reported to the manufacturer at the contact information listed on this document and the proper authority of the country or state where you are located. When treatment is complete, dispose of this device according to local laws and ordinances.

DARCO®

Body Armor® Cast Shoe**Instructions For Use**

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