Body Armor[®] Pro Term

DARCO (Europe) GmbH
Gewerbegebiet 18
82399 Raisting, Germany
Telefon +49 (0) 88 07 92 28-0 | Fax -22
info@darco.de, www.darco.de

Made in China

180608

USA/GB Indications

- > Diabetic foot syndrome
- > PAOD (peripheral arterial occlusive disease)
- > Trauma

Postoperatively for treatment following:

- > Chopart amputation
- > Lisfranc amputation

Contraindications:

- > No contraindications are known.

Technical data

2 Sizes:
M = Stump length 16 cm, L = Stump length 19 cm
Can be worn on the right or left foot.

Colour: black-blue

Material:
Outsole walking surface: TPR
Outsole body: EVA
Plastic shell: Polyurethane
Fabric liner: Polyester
Buckles: Polypropylene
Insole: EVA, PE

Maintenance:
The product is maintenance-free.

Cleaning instructions

Hand wash:
If the shoe is slightly soiled, you can wipe it with a damp cloth or clean it with a soft brush. To clean the shell and lining, use lukewarm water (30° is possible) and mild soap or mild detergent if necessary. Rinse and dry the inner shoe promptly. Do not dry the shoe in a dryer or on a radiator. Let it dry in the air at room temperature.

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Body Armor® Pro Term
Foot stump orthosis

- **Photo 1:** Open the two fasteners on the orthosis and remove the blue inner shoe. Open the hook and loop straps of the inner shoe and place the inner shoe around the leg. Fasten the inner shoe with the hook and loop straps. First close the hook and loop strap over the instep, then the lower hook and loop strap as numbered in photo 1.
- **Photo 2:** To insert the foot securely into the walker, spread the medial and lateral parts of the shell with body hands. Now place both insoles into the walker and step onto the two insoles with the blue inner shoe. The insoles can be adjusted individually by qualified staff.
- **Photo 3:** After inserting the foot, pass the straps into the buckles and adjust them to the desired pressure. Fasten the buckles.
- **Photo 4:** Check that the valve wheel is closed clockwise.
- **Photo 5:** Now pump air through the grey round pump on the front of the walker into the air chambers of the blue inner shoe. The air is distributed evenly. The degree of compression obtained should ensure that neither the stump nor the ankle can move freely. The inner shoe of the walker should secure the stump and ankle adequately.
- **Photo 6:** If impaired sensation, swelling or pain occurs, the pressure should be reduced by releasing air from the chambers. To do this, turn the valve wheel over the pump anticlockwise. Should persistent pain occur, the treatment should be interrupted and a doctor should be consulted.

Caution

Please always follow these instructions for use. The foam pad should be checked every 6 – 8 hours by qualified personnel to monitor the colour and blood circulation of the stump. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. It is advisable to wear dressings or socks with the shoe. We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual. This aid is a single-use product.

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