

Setting Medication Times (continued)

- 3. To cancel a previously set alarm, continue pressing Button B until “--:--” is displayed. This occurs after 23:XX or 11:XX pm. Or press and hold Button A until “-- : --” is displayed.
- 4. After setting a desired alarm time, confirm by pressing Button A. The next alarm number “2” will then be displayed.
- 5. Repeat above steps (2) through (4) to set alarms 2, 3, 4, 5, and 6.
- 6. After confirming alarm 6, display will return to clock mode.




Setting time format by mechanical buttons

- 1. Under clock mode, press Button C once to enter time format setting, then press Button C to switch between 12-Hour and 24-Hour time format.
- 2. Confirm selected time format by pressing Button A or just wait for approximately 15 seconds.



Setting volume control by mechanical buttons

- 1. Under clock mode, press button C once to enter volume control setting, then press Button B to switch between 0, 1 and 2. “0” means mute, “1” means low volume, “2” means high volume.
- 2. Confirm selected volume control by pressing Button A or just wait for approximately 15 seconds.

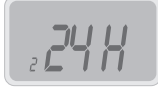

FCC Compliance Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:
(1) This device may not cause harmful interference, and
(2) this device must accept any interference received, including interference that may cause undesired operation.

FCC Warning:

§ 15.21 Information to user.
Any changes or modifications not expressly approved by the party responsible for compliance could void the user’s authority to operate the equipment.

§ 15.105 Information to the user.
This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

*** RF warning for Portable device:**
The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.

Radio Frequency

Frequency bands: 2402MHz to 2480MHzMax.
Transmission power: 8dBm

©2021 FEI, all rights reserved. CanDo,  and **FEI** are trademarks of Goldberg.

 Fabrication Enterprises Inc.
250 Clearbrook Road, Suite 240
Elmsford, NY 10523 (USA)
tel: +1-914-345-9300 • 800-431-2830
fax: +1-914-345-9800 • 800-634-5370
FabEnt.com

EC REP



Authorized CE representative:
AJW Technology Consulting GmbH
Königsallee 106
40215 Düsseldorf (Germany)

CANDo®

Automatic Pill Dispenser

User Manual (70-0285)



Contains: 1 Automatic Pill Dispenser, 4 AA Batteries, User Manual


■ Introduction

Thank you for purchasing our CanDo® Automatic Pill Dispenser (70-0285). This user manual offers you a brief walkthrough of your new device's features and usage protocols.

■ Intended Use

Whether you're using it to keep track of your own medications, or to assist or inform a family member, this pill dispenser promotes better health by alerting users and dispensing their medications at the correct times of the day and week. Organizing medication correctly, assigning dose times, and determining what medications go into each slot is the responsibility of you, your caregiver, and/or your healthcare provider.

■ Tips and Precautions


- A low battery symbol  on the screen indicates the batteries are low and should be replaced to ensure proper function. To save power, pill dispenser will stand by and display will shut down if there is no operation for several minutes. To wake the screen up, just touch or slightly move pill dispenser.
- Do not place the appliance near open flames or heat sources. Fire, electric shock, product damage, or injury might occur.
- Do not let children use or play with this device.
- Make sure to keep the pill dispenser on a flat, stable surface. Do not flip it upside down unless you are removing pills from a compartment. Please note, the internal pill container is removable for easy filling.
- Be careful not to drop the pill dispenser, as this could damage the product or the pills inside.
- Water can damage the dispenser, cause issues with the batteries, or otherwise adversely affect the medication contained inside. Do not immerse this pill dispenser in water or other liquids, or expose it to liquids that might get splashed on it. Do not place it in an environment where there is excessive moisture or humidity.
- Do not fill the internal pill container unless it is completely dry.
- Never force the locking cover open.
- Never use mismatched batteries, or batteries with wrappings that have been torn or damaged in any way.

■ Product Features

- **Automatic Pill Alerts:** The system will repeatedly flash the light indicator ring, rotate the internal pill container, and sound a beeping alarm. This will occur every seven (7) seconds for a period of up to 30 minutes, until the unit has been fully inverted.
- **28 Spacious Pill Compartments:** The internal pill container features ample space in each compartment to accommodate multiple pills or tablets, to allow you to fit all the pills you need to take at a designated time in the same slot. Pill dispenser measures 9" dia. x 2" H.
- **Clear, Removable Tray:** Lift the internal pill container out of the main body of the device for easy refilling. The container automatically rotates according to the schedule you set.
- **Locking Cover:** The powerful latch and the manual locking key help keep your medications secure and prevent messes or spills, unwanted tampering, accidental removal, or theft.


■ Parts & Controls

1. Top Cover
2. Cover Handle
3. Dispenser Door
4. Dosage Viewer
5. Light Indicator
6. Dosage Ring
7. LCD Display
8. Button A
9. Button B
10. Button C
11. Removable Tray
12. Cover Lock
13. Battery Door
14. Battery Door Latch
15. Speaker
16. Cover lock

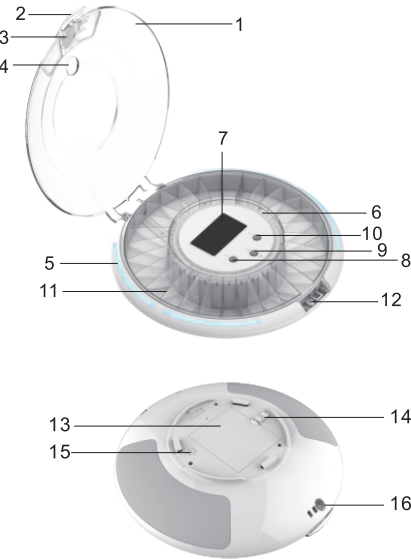


■ Inserting Batteries

1. Turn the dispenser upside down.
2. Slide Battery Door Latch and remove Battery Door.
3. Insert 4 AA batteries in correct polarity and replace door. LCD Display will turn on.

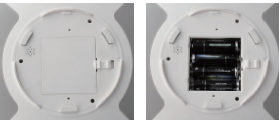

■ Open or Lock Top Cover


1. If locked, locate Cover Lock on bottom of unit and insert the included key, then turn clockwise to unlock.
2. Put pill dispenser on table, press Cover Handle to release the latch, then lift to open Top Cover.
3. To lock Top Cover, close Top Cover first, press Cover Handle to latch, then turn counterclockwise to lock by inserting the key from the bottom side.

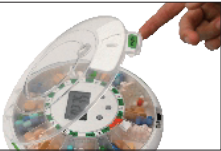

■ Place the Dosage Ring

1. Automatic pill dispenser comes with 9 different rings, corresponding to the number of times per day that medication needs to be dispensed. The rings are designed to help you lay out medication in the internal pill container so it is properly dispensed.
2. Set correct Dosage Ring on center of tray, and fold the tab down to secure in place.
3. Place pills in correct dosages in each compartment.

* Do not place pills in compartments corresponding to red markings on Dosage Ring.

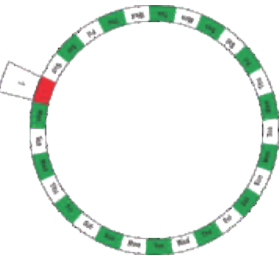


■ Take Medication and Stop Alarm

Pill dispenser will alarm when it's time to take medication. Slide to open the Dispenser Door and tilt pill dispenser to pour medications into your palm or a suitable container. The alarm and Light Indicator will then shut automatically.



■ Low Battery & Power Saving

A low battery symbol  on the screen indicates the batteries are low and should be replaced to ensure proper function. To save power, pill dispenser will stand by and display will shut down if there is no operation for several minutes. To wake the screen up, just touch or slightly move pill dispenser.



■ Setting with Mechanical Buttons

Open the Top Cover first as stated in the "Open or Lock Top Cover" section.

Setting the Time

1. Press and hold button A for approximately 3 seconds until ":" symbol stops flashing.
2. Press button B to adjust hour digits. Press button C to adjust minute digits. * Pressing and holding buttons B or C will cause the digits to speed up.
3. Confirm clock settings by pressing button A.



Setting Medication Times

1. Press button A once, an "alarm clock" symbol and number "1" will be displayed.
2. Press Button B to adjust hour digits. Press Button C to adjust minute digits. * Pressing and holding buttons B or C will cause the digits to speed up.

